You can screen for cervical cancer at home:

Step-by-Step Instructions for a Vaginal Swab


One way to test for HPV at home is by using a vaginal swab. If you have questions, or think you made a mistake, call the **Unidos contra el VPH** team (915-308-1148) or **Project Vida** (915-465-1191). We are here to help you even if you are testing at home.

Please read all the steps before you start. Avoid distractions. Once you start the process, finish all the steps.

1. Wash your hands

Wash your hands with soap and water. Don't touch things that you don't need for the test.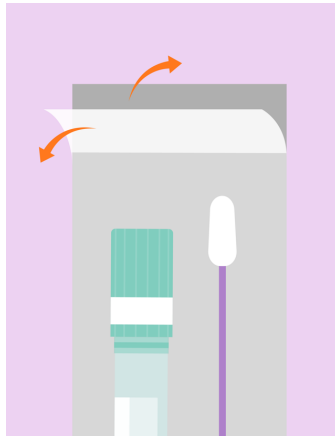


2. Open packaging

Tear open the paper envelope at the top, away from where the expiration date is printed. Inside you'll find the swab and tube.


3. Take out the swab

Open the package where it says, "Peel here" and pull out the swab. Don't touch the cotton tip of the swab.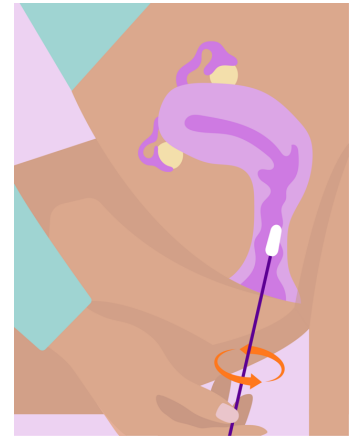


4. Swab the vagina

Gently insert the swab about the length of a finger. Turn the swab in circles for about 30 seconds and make sure the swab touches the walls of the vagina. Then, remove the swab. Don't set the swab down; keep it in your hand.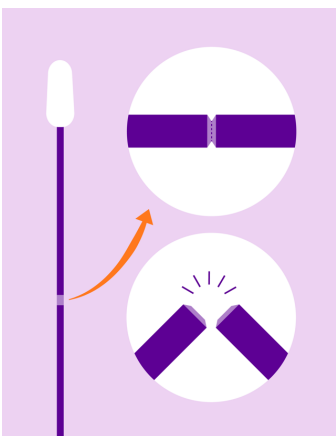

5. Break the swab

The swab is long so you need to break it so it fits in the tube. Break the swab where the red dot is. You should be able to snap the piece off.

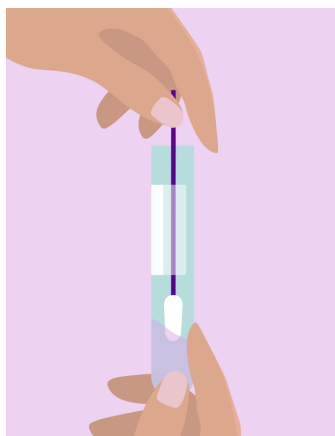6. Put the swab in the tube

Hold the swab and with the same hand unscrew the cap from the tube. Put the cotton tip of the swab into the red liquid inside the tube.

7. Check the cap is screwed on tight

The cap must be on tight. Turn the cap to the right until it won't turn anymore. Turn the tube upside down, over the sink, to check the cap is on completely.

8. Put sample tube inside of orange biohazard bag

Seal the biohazard bag once your sample is safely in the bag. This will be like closing a Ziploc bag.

Packing your test sample for mailing

Step 1: Take the packing slip from inside of the box and stick it to the outside of the box.

Step 2: Put biohazard bag with sample tube inside the box.

Step 3: Close and seal the prepaid FedEx box.

Step 4: Keep your tracking information.

Returning your test

We want to make it easy to return your test sample. There are two ways to return the test.

Option 1: Call the Unidos Contra el VPH team (915-308-1148) to schedule a pick-up from your home.

- You can call or text our team at 915-308-1148.
- Please tell us:
 - When you'll be home
 - Address (including apartment number if you have one)
 - Any other info to help us find your home

Option 2: Drop off your sample at a FedEx store.

- The box already has paid postage.
- You just need to drop off the box at the FedEx counter.

What to expect after mailing your sample

You'll get your test results in about two weeks. Our team or your Project Vida doctor will call you with the results.

If you have any questions, please call the Unidos Contra el VPH team (915-308-1148).

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Step-by-Step Instructions for a Urine Test

One way to test for HPV at home is by giving a urine sample. If you have questions, or think you made a mistake, call the **Unidos contra el VPH** team (915-308-1148) or **Project Vida** (915-465-1191). We are here to help you even if you are testing at home.

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
2. Urinate in the cup

The urine test can be done at any time of day. The most important thing is to collect the sample at the start of urination, because that's when the most HPV can be detected. Try to fill the cup to the 20mL mark.

3. Wash your hands

Check the cup cap is screwed on tight. The cap must be on tight. Turn the cap on the cup until you hear a clicking sound. Once it clicks the cup will be secure for mailing.

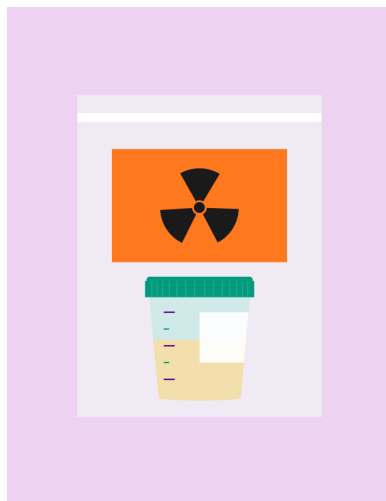4. Put sample cup inside of the orange biohazard bag

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